

## NAVIO 6.0: TKA (Distal Bur)

A fast, efficient technique that uses a roboticassisted handpiece to prepare the distal femur in total knee applications (TKA).



# NAVIO°: The **only handheld** robotics technology system for TKA, UKA and PFJ applications

JOURNEY° II, LEGION°, GENESIS° II: The largest knee portfolio enabled by handheld robotics

The NAVIO Surgical System is an intraoperative visualization and surgical planning system combined with a handheld, smart instrument for bone sculpting. Through the use of a robotic-assisted handpiece, the NAVIO system limits the amount of bone that is removed.

### NAVIO 6.0

#### **Distal Bur Workflow (TKA)**

The distal bur technique features robotic-assisted preparation of the distal femur and allows surgeons to set rotation by setting two pilot holes for the AP cut block.

#### Femur preparation

**Step 1:** The distal femur is prepared using a surgeon controlled, robotic-assisted handpiece and a 5mm bur.






**Step 2:** Two peg holes are drilled with a robotically controlled 2mm bur and the AP cut block is placed.



**Step 3:** Chamfer cuts are made through a familiar approach.

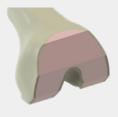

#### **Tibia** preparation

**Step 4:** The tibia is prepared using a robotically controlled 5mm bur that drills post holes for the NAVIO cut guide.



**Step 5:** Place the NAVIO cut guide in robotically prepared holes and perform the bone resection.





#### **Familiar workflows**

NAVIO° integrates into familiar surgical workflows (femur first, tibia first, spacer block technique) with minimal disruption to current total knee approaches. Surgeons can choose one of three robotic-assisted workflows [cut guides, distal burring (hybrid saw and bur), bur all].







Cut guide workflow

Distal bur workflow

Bur all workflow

#### Image-free system

No preoperative CT required – for less exposure to radiation and eliminated costs associated with other image-based systems.

#### Dynamic ligament balancing throughout the procedure

Enables the surgeon to assess soft tissue and balance the knee throughout the full range of motion (ROM).





#### Patient-specific plan

Created using state-of-the-art software that provides a unique, intraoperative 3D model based on the patient's anatomy.

#### Small, compact footprint ideal for ASC and Hospitals

Easily move NAVIO room to room or facility to facility.



#### For more educational videos and content, go to:

#### **Smith & Nephew Education & Evidence**

smith-nephew.com/education

#### SmithNephewlivesurgery.com

